

Release Notes v. 2023.2.1*

What we've fixed

We fixed the bugs below to make your CAD experience better.

Abutment post height

We temporarily disabled the minimum 4 mm abutment post height requirement due to negative user feedback. We are currently working with the FDA directly to better clarify this requirement and will keep you up to date.

Model disappearing issue

Fixed a bug that caused the 3D model to disappear when adding a serial mark containing certain characters, particularly those from Asian fonts.

Wax-up sleeve holder

Resolved an issue that prevented the wax-up sleeve holder from being used.

*This version is compatible with Dental Wings standalone software licenses and the following lab scanners:
7Series, 3Series, and Medit T500, T310, T510, and T710.