

STRAUMANN SIRIOS™

Installation Instructions v 1.0



Table of Contents

Table of Contents	2
1 Introduction	3
2 Prerequisites	4
2.1 Check your system requirements	4
3 Installation	5
3.1 Download Straumann SIRIOS	5
3.2 Install Straumann SIRIOS	6
4 Registration	8
4.1 Request registration	8
4.2 Finish registration	10

1 Introduction

Use this document for help installing Straumann SIRIOS™ software on your PC.


In this document you will find:


- the minimum computer requirements for Straumann SIRIOS,
- how to install Straumann SIRIOS, and
- how to register Straumann SIRIOS.

2 Prerequisites

2.1 Check your system requirements

The table below contains the system requirements for Straumann SIRIOS installation and operation.

 **Note:** Do all Windows updates before starting the installation.

 **Warning:** If you have other Straumann IOS software installed, uninstall it before installing Straumann SIRIOS software to avoid unexpected behavior.


	Minimum requirements	Recommended requirements
Operating system	MS Windows 10 (build 18362+) MS Windows 11 - 64 bit	MS Windows 10 (build 18362+) MS Windows 11 - 64 bit
Processor (desktop)	9th Generation Intel Core i5-9600 AMD Ryzen 5 2600	9th Generation Intel Core i7-9700 AMD Ryzen 7 2700
Processor (Laptop)	10th Generation Intel Core i7-10750H AMD Ryzen 7 4700U	10th Generation Intel Core i7-10870H AMD Ryzen 7 PRO 4750U
Memory (RAM)	16 GB	32 GB
Graphic card model	NVIDIA GeForce GTX 1650 4GB	NVIDIA GeForce GTX 1660 6GB
Graphic card drivers	Update to the latest version available from the manufacturer's website.	Update to the latest version available from the manufacturer's website.
Storage	500 GB SSD	1 TB SSD

3 Installation


Follow these steps to download and install Straumann SIRIOS software.

3.1 Download Straumann SIRIOS

1. Click [here](#) to download Straumann SIRIOS. Wait for the download to complete.
2. Go to **Downloads** and click **Open file**.

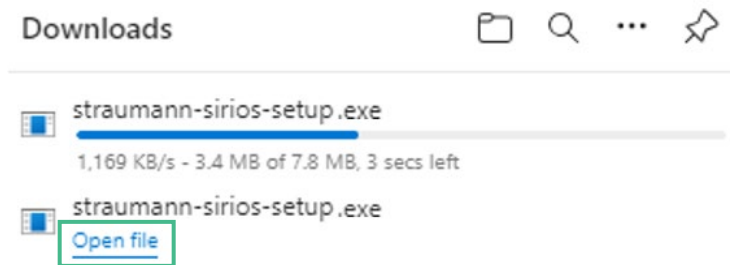


3. The installer downloads automatically. This takes a few minutes.

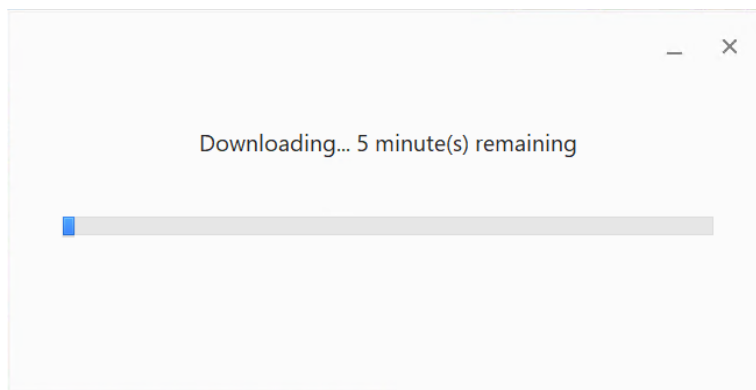


3.2 Install Straumann SIRIOS


1. When the download is complete the installer opens. Click **Next**.

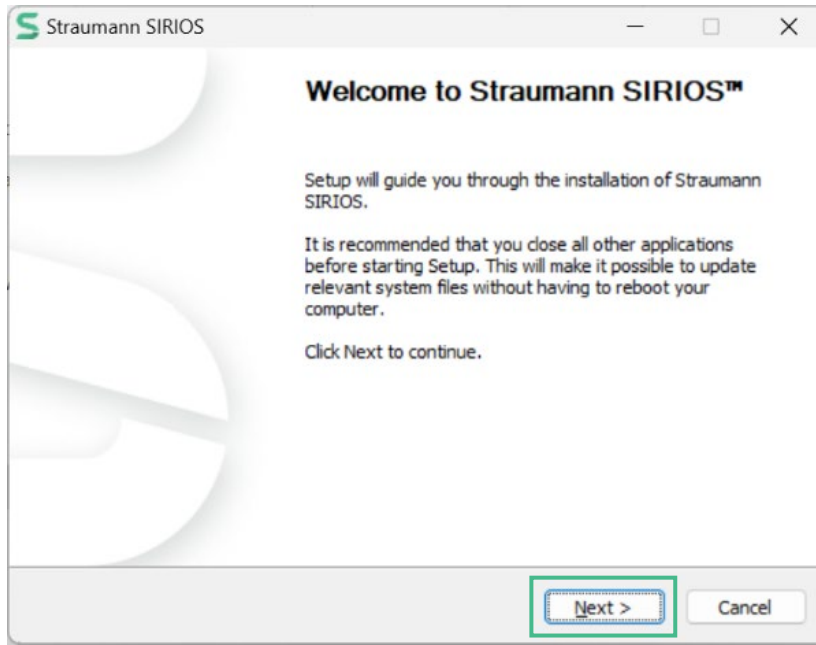

2. Click **Install**.

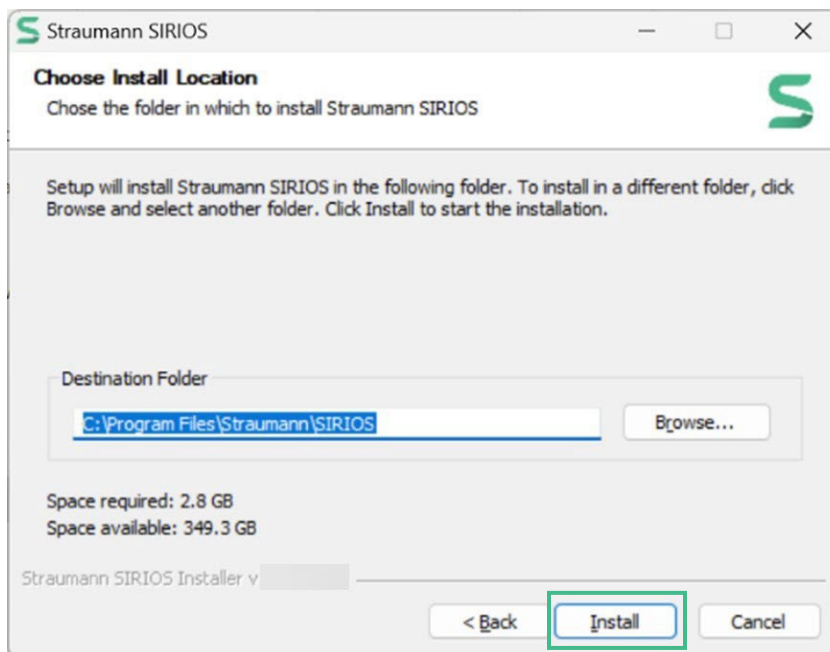

3. Click **Close** when the software has finished installing.

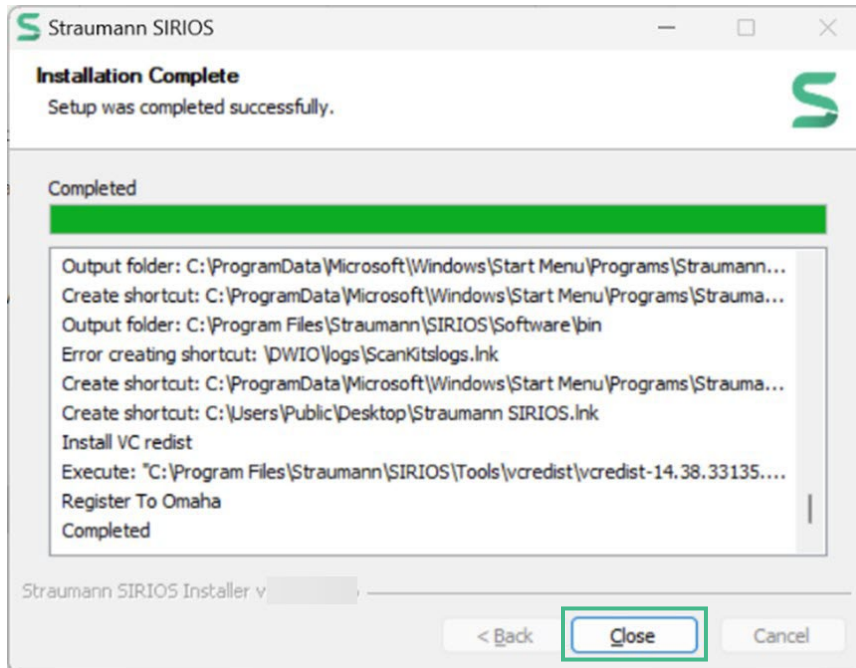


4 Registration


Follow these steps to register Straumann SIRIOS.

4.1 Request registration

1. Double-click the **Straumann SIRIOS** icon on your desktop icon to start it.

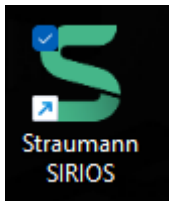


2. Wait for the *Product registration* window to appear.
Click **Next**.

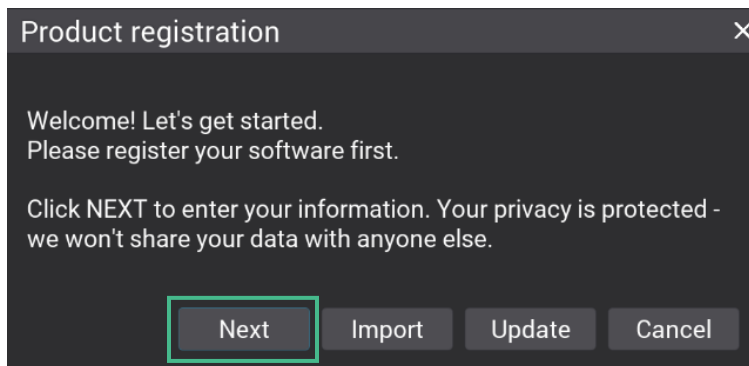

3. Fill in the *Registration information*, making sure all information is complete and accurate.

⚠ Warning: Do NOT use the Register button. This feature is coming soon but using it now can lead to unexpected behavior.



📖 Note: Use the Info box to add any additional information (e.g. you replaced your laptop, the need is urgent, or to add information for an additional contact).

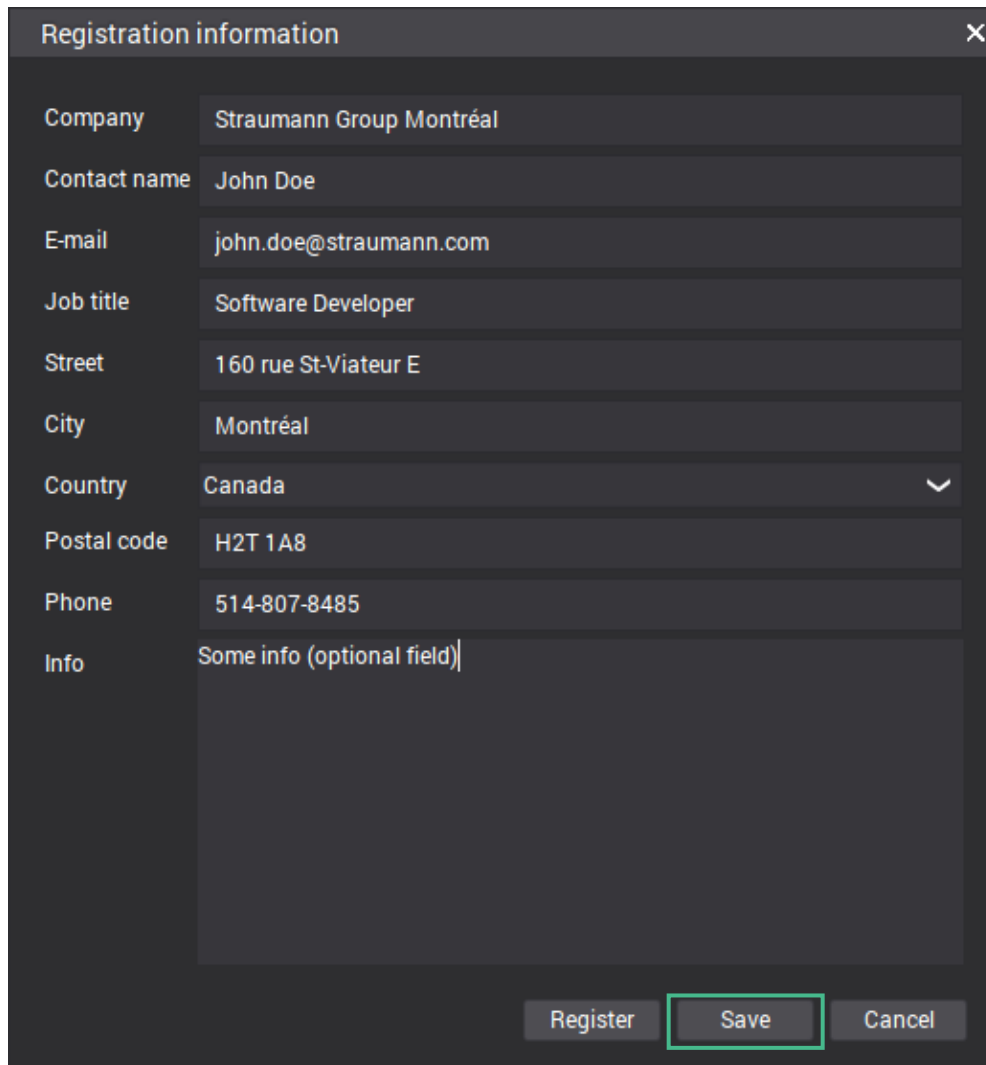

The image shows a dark-themed dialog box titled "Registration information" with a close button (X) in the top right corner. The dialog contains several input fields, each with a label on the left and a text input area on the right. The fields are: Company (Straumann Group Montréal), Contact name (John Doe), E-mail (john.doe@straumann.com), Job title (Software Developer), Street (160 rue St-Viateur E), City (Montréal), Country (Canada, with a dropdown arrow), Postal code (H2T 1A8), Phone (514-807-8485), and Info (Some info (optional field)). At the bottom of the dialog, there are three buttons: "Register", "Save", and "Cancel". The "Save" button is highlighted with a red rectangular border.

4. Click **Save** and then select the location to save the registration request.


5. Click **OK** in the *Information* window and then **OK** for No valid license found.

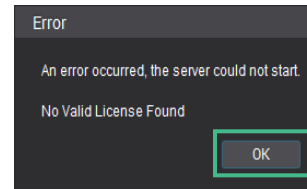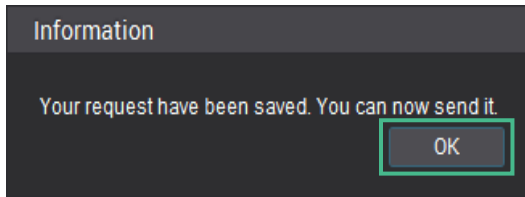


6. E-mail your saved registration request to license-request@dental-wings.com.
7. The registration team will e-mail you a registration file (.lic) within 24 weekday hours. Save it to the laptop.

4.2 Finish registration


1. Double-click the **Straumann SIRIOS** icon on your desktop.

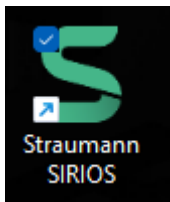


2. Wait until the *Product registration* window appears. This may take a few minutes.
3. Click **Import**.

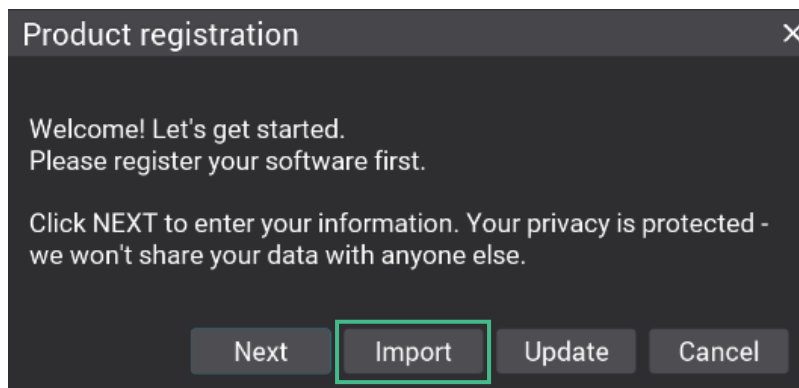


4. Confirm the import action.

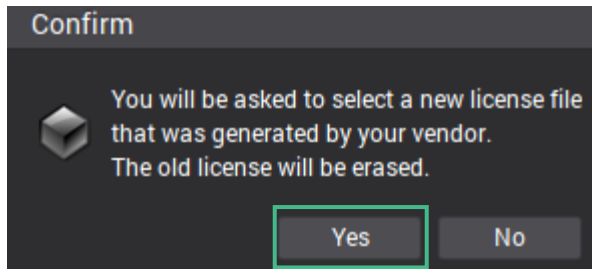


5. Select the .lic file you saved in step 7 above.

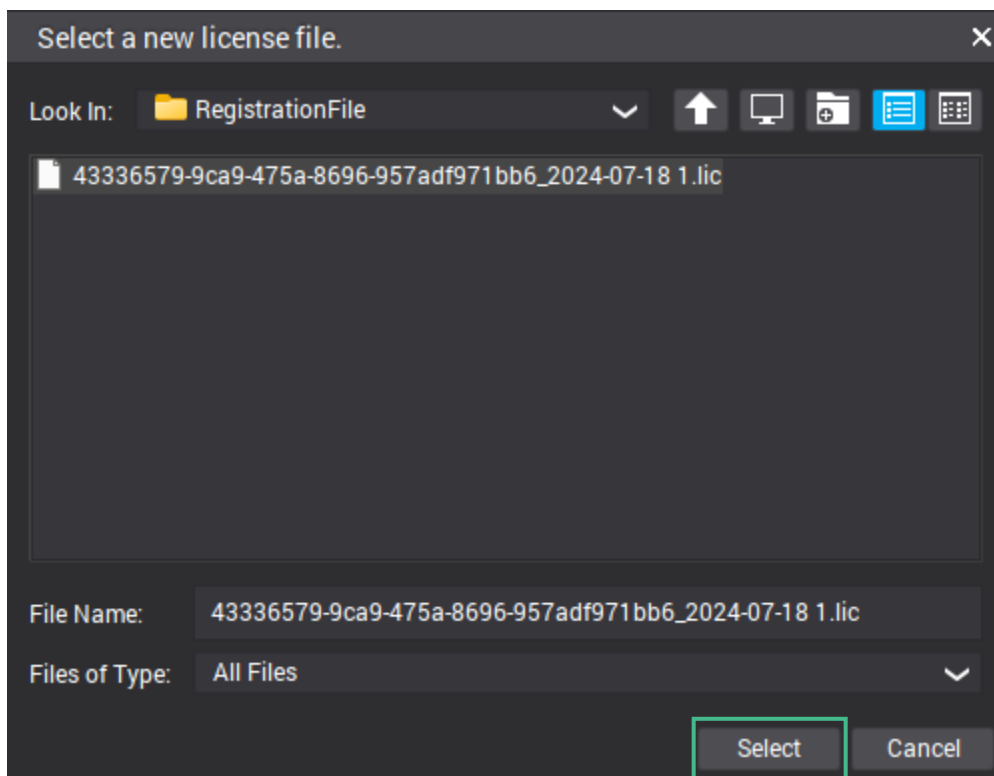

6. Confirm your file selection

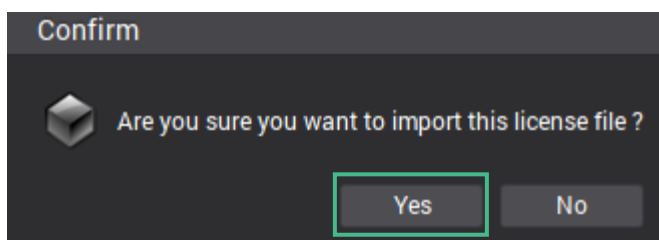

7. Click **OK** to start Straumann SIRIOS.

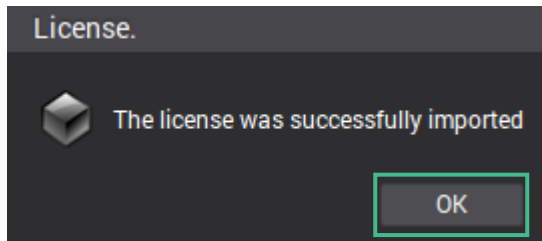

8. If the *Location* window opens, click **Yes**.

